



CareCam Wall

On-Site Installation Instructions







Parts Overview



Camera Module



Intercom/Power Module



GCX Channel, Stops, and Wire-Shroud *GCX Channel should already be installed on customer's wall and will generally not be included.





Assembly



Intercom / Power Module

Slide a stop onto the GCX channel to the desired height for the Intercom/Power module and screw on until snug



If power is coming from above the Intercom/Power module, route the power cable down the channel

If ethernet is desired below the Intercom/Power module, route the ethernet cable down the channel



Slide the Intercom/Power module onto the channel and let it rest on the stop.







Any in-place wires should go under the module's mounting bracket



2

Camera Module

Place a stop at the desired height for the camera module and screw it until snug


The ribbon cable can be routed under the stop



Gently route cables that are exiting the top of the camera down under the mounting bracket

Slide the camera module onto the channel until it rests on the stop

Connect the ribbon cable, matching colors to like colors

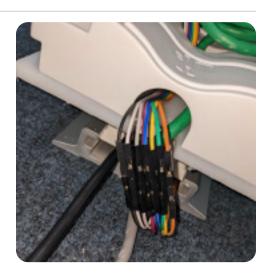

CareCam Wall

On-Site Installation Instructions





Excess ribbon or ethernet can be tucked inside the camera-arm housing by removing the cover and carefully pulling it inside



Plug the power cable into the socket at the bottom of the intercom/power module

Plug a patch cable into the female ethernet lead into the desired wall jack

Power on the unit

